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Nonle User Interface - DEFAULT PROFILE

File Settings Monitors CIM Box CPR Help

Status Details Palette Scenario Labor Log

Gaumard®

Clear Settings Load Palette Item... Save as Palette Item...

Other (Temperature: °C)

Breathing Respiratory Pattern: (OSel) ☐ % ☐ % change
Respiratory Rate: breaths / min ☐ % change ☐ mmHg ☐ % change
Inspiratory Time: %

Circulation Cardiac Rhythm: **Blood Pressure** Systolic: Diastolic: ☐ mmHg ☐ % change

UA Contraction Frequency: min Contraction Duration: sec Contraction Intensity: mmHg
Resting Tone: mmHg Coupling: % Probability % Size

FHR FHR Baseline: bpm Variability: Accel/Decel Intensity:
Episodic Changes: Periodic Changes: Variable Changes:

Apply

NOW 10 sec 30 sec 1 min 2 min 5 min 10:00 min

Clocks Transition remaining: 00:00:00 Session: 00:02:28

Evaluation Care Provided: Unsatisfactory Satisfactory
Note: <- Add to log

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Moelle User Interface - QUICK START SCENARIOS

File Setup Monitors CIM Box CPR Help

Status Details **Palette** Scenario Labor Log

Gaumard®

Status

Vitals Petal Mon.

Breathing

Resp. Pattern: **normal**

Rate: 13 rpm in 33 %

(O2-Sat: 98 %)

(EtCO2: 40 mmHg)

Circulation

Cardiac Rhythm: **Sinus**

Heart Rate: 70 bpm

Blood Pressure: 120 / 80 mmHg

(Temperature: 37.5 °C)

Name	Description
Alice 1	Admission
Alice 2	Transitional Labor
Alice 3	Second stage
Alice 4	Delivery
alice01	Stage1 - Active
alice02	Stage1 - Transition
alice03	Stage2
alice04	Stage3
Alicia 1	Admission
Alicia 2	Transitional Labor
Alicia 3	Second stage
Alicia 4	Delivery
Amy 1	Admission
Amy 2	Early Labor
Amy 3	Active Labor
Amy 4	Transition
Amy 5	Pushing

View

Sort by Name

Healthy

Care Required

Critical

Other

Selected Item

Edit...

Delete

Properties

Apply

NOW 10 sec 30 sec 1 min 2 min 5 min 10:00 min Edit

Clocks

Transition remaining: 00:00:00

Session: 00:14:13

Evaluation

Care Provided: Unsatisfactory Satisfactory

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Noelle User Interface - QUICK START SCENARIOS

File Setup Monitors CIM Box CPR Help

Status

Vitals Fetal Mon.

Breathing

Resp. Pattern: normal

Rate: 20 bpm in 33 %

(O2-Sat: 98 %)

(EtCO2: 40 mmHg)

Circulation

Cardiac Rhythm: Sinus

Heart Rate: 80 bpm

Blood Pressure: 114 / 80 mmHg

(Temperature: 37.0 °C)

Details Palette Scenario Labor Log

Clear Settings Load Palette Item... Save as Palette Item...

Alice 1

Breathing Respiratory Pattern: normal (OSat) 98 % change (EtCO2) 40 mmHg change

Respiratory Rate: 20 breaths / min % change Inspiratory Time: 33 %

Circulation Cardiac Rhythm: Sinus options: Blood Pressure: 114 / 80 mmHg change

UA Contraction Frequency: 3 min Contraction Duration: 60 sec Contraction Intensity: 60 mmHg

Resting Tone: mmHg Coupling: 0 % Probability 0 % Size

FHR FHR Baseline: 140 bpm Variability: moderate Accel/Decel Intensity: average

Episodic Changes: reactive Periodic Changes: none Variable Changes: none

Apply

NOW 10 sec 30 sec 1 min 2 min 5 min 10:00 min Edit...

Clocks

Transition remaining: 00:00:00

Session: 00:39:49

Evaluation

Care Provided: Unsatisfactory Satisfactory

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Noelle User Interface - QUICK START SCENARIOS

File Setup Monitors CIM Box CPR Help

Status

Vitals Fetal Mon.

Breathing

Resp. Pattern: **normal**

Rate: 13 bpm in: 33 %

(O2-Sat: 98 %)

(EtCO2: 40 mmHg)

Circulation

Cardiac Rhythm: **Sinus**

Heart Rate: 70 bpm

Blood Pressure: 120 / 80 mmHg

(Temperature: 37.5 °C)

Details Palette Scenario Labor Log

Clear Load Scenario... Save Scenario... Scenario Auto-Responses... Go to Branching Scenario

Add to scenario...

Name	Transition	Description
------	------------	-------------

Selected Item

Move

Edit Transition

Insert...

Remove

Duplicate

Properties

Clocks

Transition remaining: 00:00:00

Session: 00:05:54

Evaluation

Care Provided: **Unsatisfactory** **Satisfactory**

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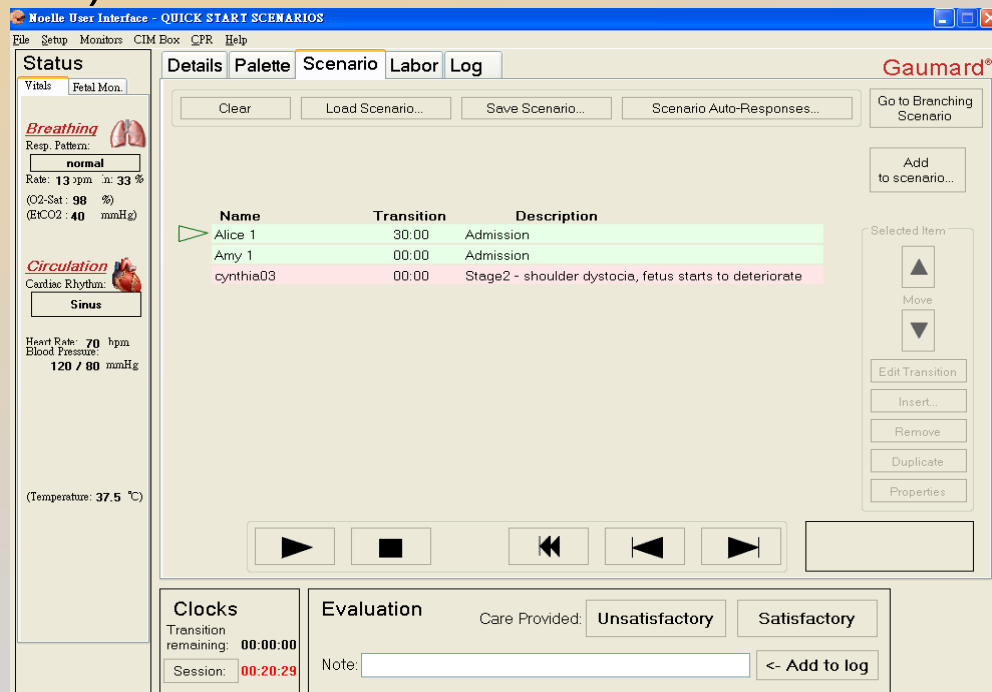
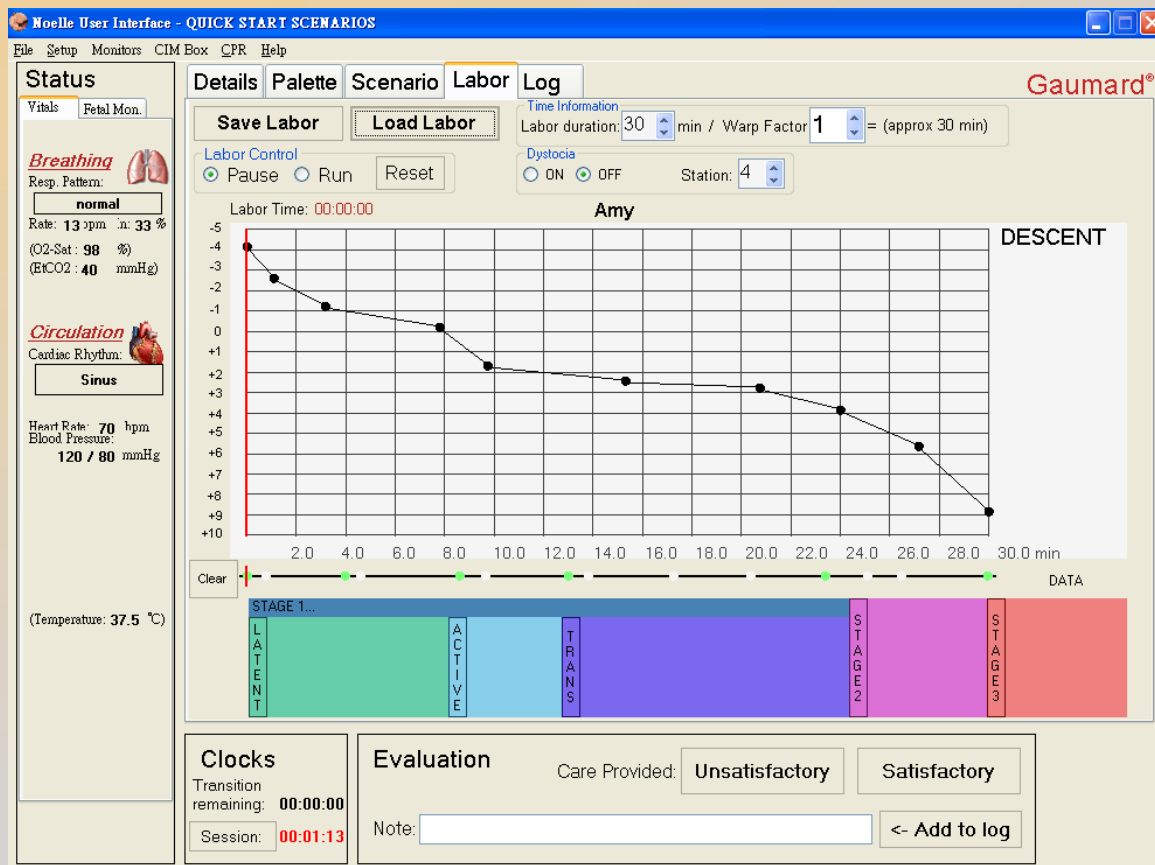
Load Cancel Delete

Patient Information

Alice is a 24 year old female, weighing 170 pounds. Her companion's name is Mike.

Her OB history shows a gravida of 2, a term of 1, a preterm of 0, and that she is currently 39 weeks pregnant; she has had 0 spontaneous abortions, 0 elective abortions and 1 living children.

She has been currently using no medications of any kind.

Her medical records shows she has: no previous history.

Her treatment history includes:NO.

S1 - Latent S1 - Active S1 - Transition Stage 2 Stage 3

Mother Info:

- ☐ Bleeding
- ☐ Cephalic Disproportion
- ☐ Amniotic Fluid Embolism
- ☐ Uterine Rupture
- ☐ Cord Presentation
- ☐ Fever

Pain Shock Airway Breathing Circulation AVPU Other

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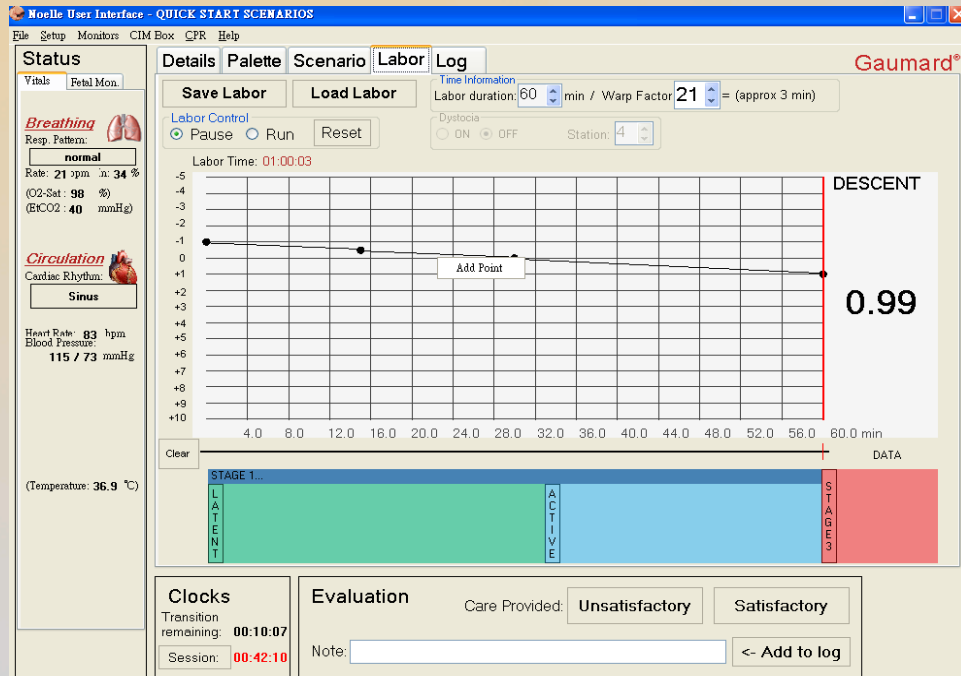
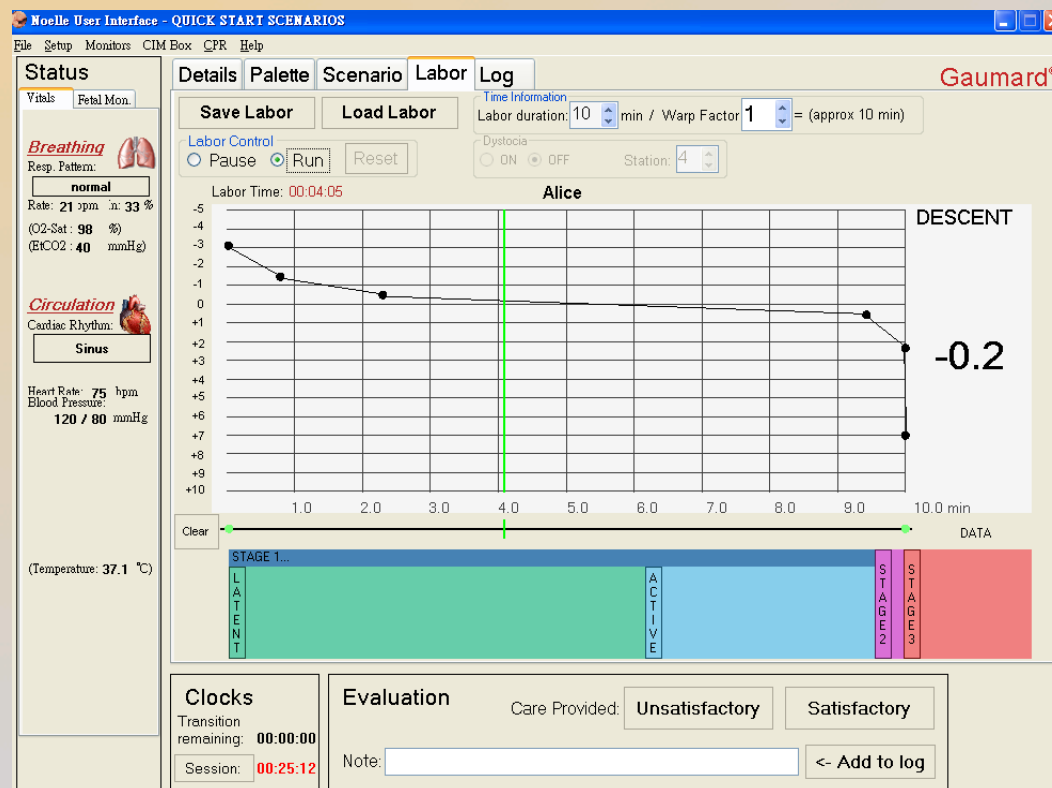
Providers: Team

Emergency Reported

- Assess responsiveness
- Call for assistance +
- Shoulder dystocia +
- Manage bleeding +
- Shock +
- Malpresentation +
- Seizures +
- Cord prolapse +

Airway

- Determine patency
- Open airway +
- Airway management +
- Intubation check +
- Ventilate +
- Extubate >

Breathing

- Assess breathing +
- O2 device +
- O2 flow +
- Wheezing / Rales
- Pallor

Circulation

- Vagal maneuver +
- Attach electrodes +
- Check for pulses +
- Interpret rhythm
- Establish IV
- Cyanosis

Medication

Drug: Aspirin Dose: Units: mg Route: PO

00:02:32 Note: hihi
00:02:48 Note: how about you
00:02:58 Action (Call for assistance)
00:03:04 Action (Call for assistance): call for AED
00:03:18 Action (Assess responsiveness)
00:03:58 Note: 誰來幫一下

Clocks
Transition remaining: 00:00:00
Session: 00:04:05

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Name of Companion: Mike

OB History:

2	Gravida	0	Spont. Abortions
1	Term	0	Elective Abortions
0	Preterm	1	Living Children
39	Gestational Age		

☐ Tobacco	☐ Diabetes	☐ Rubella
☐ Alcohol	☐ Hytension	☐ HIV
☐ Drug Use	☐ Anemia	☐ Mumps
☐ Prescriptions	☐ Tuberculosis	☐ STD
☐ Allergies	☐ Heart Disease	☐ Chicken Pox
☐ Previous C/S	☐ Group B Strep	

Treatment History: ☒ None

☐ Oral Airway	☐ Nasal Airway
☐ Nasal Cannula O2	☐ Face Mask O2
☐ Endotracheal Tube	☐ Nasogastric Tube
☐ Bag-Valve-Mask	☐ CPR

Labor type / Other:

Stage Information:

S1 - Latent | S1 - Active | S1 - Transition | Stage 2 | Stage 3

Mother Info:

☐ Bleeding	☐ Uterine Rupture
☐ Cephalic Disproportion	☐ Cord Presentation
☐ Amniotic Fluid Embolism	☐ Fever
Pain	
Shock	
Airway	
Breathing	
Circulation	
AVPU	
Other	

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